

HAEVOLANCE[®] Plus

Safety, efficacy and comfort in two simple steps

CE 0344

Features and benefits

SAFETY

- Single-use only
- automatic needle retraction and lock out
- needlestick injury and cross-infection prevention

EFFICACY

- wide product range – available in 6 different sizes
- derives proper amount of blood for different capillary tests
- dedicated versions for heel stick blood tests on neonates

COMFORT

- easy to use in just two steps
- design to control needle depth



HAEMOLANCE⁺ Plus[®]

Safety, efficacy and comfort in two simple steps



Paediatric

1,5 mm blade
1,2 mm depth

Cat. no. 7375 / 100ct box
Cat. no. 7365 / 200ct box



Max Flow

1,5 mm blade
1,6 mm depth

Cat. no. 7374 / 100ct box
Cat. no. 7364 / 200ct box



High Flow

18G needle
1,8 mm depth

Cat. no. 7373 / 100ct box
Cat. no. 7363 / 200ct box



Normal Flow

21G needle
1,8 mm depth

Cat. no. 7372 / 100ct box
Cat. no. 7362 / 200ct box



Low Flow

25G needle
1,4 mm depth

Cat. no. 7371 / 100ct box
Cat. no. 7361 / 200ct box



Micro Flow

28G needle
1,6 mm depth

Cat. no. 7370 / 100ct box
Cat. no. 7360 / 200ct box

INSTRUCTIONS FOR USE:



Twist off the protective cap, and then pull it straight out.



Place HAEMOLANCE PLUS[®] on the desired puncture site and press the button to activate the device.



Gently apply intermittent pressure near the puncture site to obtain the required blood volume.

HTL STREFA
high tech lab

Manufactured by HTL-STREFA S.A.
Adamówek 7, 95-035 Ozorków,
Poland
Tel. +48 22 587 05 72,
Fax +48 22 811 05 36
info@htl-strefa.pl, www.htl-strefa.pl



Contact in the US
HTL-STREFA, Inc.
3005 Chastain Meadows Pkwy, Suite 300,
Marietta, GA 30066, USA
Tel +1 770 528 0410,
eFax: +1 770 872 0513
info@htl-strefa.com, www.htl-strefa.com